

BD EleVation™

Breast Biopsy System

Demo Probe Reset Quick Reference Guide

If the probe becomes misaligned during demonstrations, you won't be able to insert in into the BD EleVation™ Driver. You will need to readjust the main assembly and sample container.


Main Assembly

Sample Tank




1

Push the sample tank to the back of the probe.

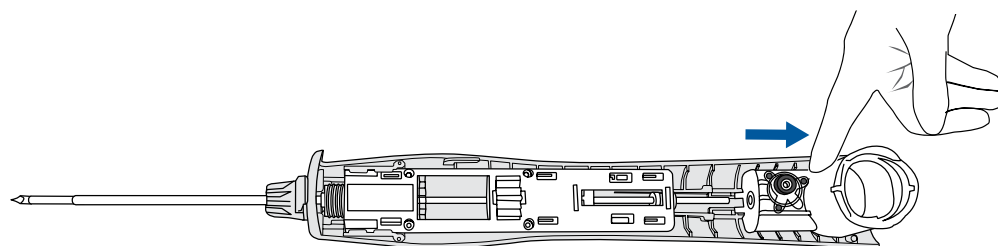

2

Push the main assembly towards the sample tank. You will hear a click when it is in the correct position.

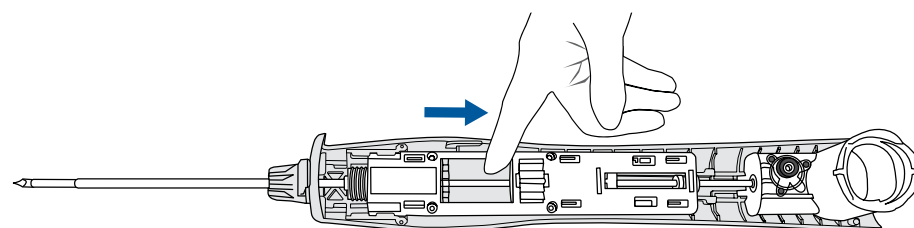

TIP:

You can use the retainer which comes with the probe tray to see exactly where each part fits.



BD Switzerland Sarl
Terre Bonne Park – A4, Route De Crassier, 17, 1262 Eysins, Vaud, Switzerland
T: +41 21 556 3000

CE
2797

bd.com

BD, the BD Logo and EleVation are trademarks of Becton, Dickinson and Company or its affiliates.
©2021 BD. All rights reserved. 0321/6126 BD-28407

